


AirSalter™

Getting to know your device

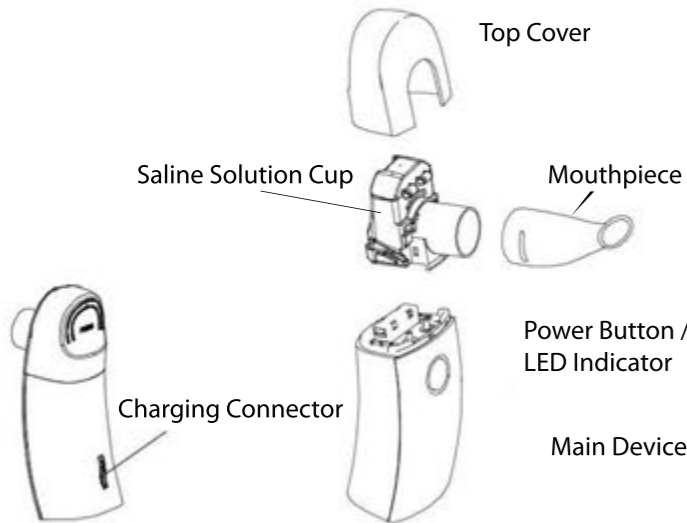

Operating Instructions

- Before using the AirSalter™ for the first time, charge the device for 4 hours.
- Open and remove the transparent top cover and then open the saline solution cup cover.
- Fill the solution cup with at least 2 ml (up to 6 ml) of Dead Sea Saline Solution and close the cover.
- Put the top cover back and make sure it's in all the way, or the device won't turn on.
- Attach your mouthpiece or face mask if needed.
- Click the power button - the indicator light will turn blue and mist will start coming out. Keep the device at vertical position throughout the session.
- The device will turn off automatically after 10 minutes or when the saline solution cup is empty.
- A flashing green light indicates low battery, charge

until a solid blue light appears, indicating device is fully charged (usually 3-4 hours).

Cleaning Your Device

It is extremely important to clean the AirSalter™ after each use. Here is how to do so:

1. If any saline solution is left in the device, empty it out.
2. Fill only the cup with tap water and turn on the device, allowing it to run for one minute. The water rinses and clears the device's system from the saline solution.
3. After a minute, empty the cup and allow the device to air dry.
4. Gently wipe the round metal vent on the front of the device with a dry Q-tip.

*** Do not rinse the device or submerge it in water.**

Use Recommendations

- The AirSalter's default setting is programmed for 10-minute sessions.
- The recommended number of daily sessions is two, for as long as the health condition persists.
- For faster results or more severe conditions, it is safe to do three or four sessions per day.
- Use for respiratory conditions: Inhale slowly and deeply, hold your breath briefly for 5 to 10 seconds and then exhale quickly. The right breathing technique is important to ensure that the particles are distributed evenly into the air ways.
- Use for skin conditions: Expose your skin where treatment is needed and place the AirSalter™ close to this area. There is no need to use any of the accessories (mouthpiece, masks).

Typical Specifications

- Power: DC 3.7V Lithium Batteries
- USB Charger: 5.0V 1.0A
- Size: 54mm x 48mm x 135mm
- Weight: 5.3 oz / 150 g
- Solution Capacity: 6 ml
- Protection: Class II, Type B Applied Part
- Ultrasonic Frequency: 110KHz±10KHz
- Noise Level: Less than 50dB
- Vapor Ratio: 0.15ml/min - 0.90ml/min
- Mist Particle Diameters: Over 50% of 1um~5um
- Storage Temperature: 32°F - 104°F / 0°C-40°C
- Continuous Working Time: Up to 1 hour
- Country of Manufacture: China
- Warranty: 90 Day Limited Warranty



care@healovation.com
(888) 373-4325

AirSalter™

We've all done it. We get to the beach and we take a deep, deep breath. It's a reflex. It's instinctual. The body knows to breathe this air in before the mind realizes it's even out of the car.

Here's what our bodies know and what our minds need to be reminded of – Breathing aerosolized salt (tiny salt particles suspended in air) can provide broad health benefits.

Allergies, chronic respiratory illnesses, skin disorders, infections and problems sleeping can all be improved by simply breathing while relaxing in the wonderful healing microclimate of the AirSalter™.

Salt Therapy Benefits

Salt Therapy has been reported effective in treating a variety of upper and lower respiratory tract disorders and skin conditions such as asthma, bronchitis, emphysema, cystic fibrosis, eczema and psoriasis.

The salt particles can help to open up the airways, increase oxygen intake and cleanse the airways of smoke, dust, pollutants and other allergens. When the salt lands on the skin it can also help to relieve sore, flakey, red and irritated skin.

It is important to understand that salt therapy is not an alternative therapy, but a complementary one. It can help relieve the symptoms of a condition and work in conjunction with existing medication, resulting in even greater relief.

90-Day Limited Warranty

Your new AirSalter™ device is covered by a limited 90-day warranty.

We will repair or replace any product that malfunctions due to a manufacturing defect within three months of purchase.

The limited warranty does not cover any physical damage or any damage caused by:

- Placing the AirSalter™ device in a dishwasher.
- Submerging the device under water.
- Dropping the device.
- Using any solution other than the AirSalter™ Dead Sea Saline Solution.
- Exposing the device to extreme heat (such as

leaving the device in the car, or in direct sunlight for an extended period of time).

- Any other reckless or irresponsible use.

Please note: The AirSalter™ device is not waterproof and should not be rinsed or submerged in water. The AirSalter™ device must be used only with Healovation's saline solution in order for the warranty to be in effect. Using any other saline solution, or other liquid in the device will VOID the warranty.

DISCLAIMER: Statements regarding this product have not been evaluated by the FDA, and this product is not intended to diagnose, treat, cure, or prevent any disease or condition. Salt Therapy is not an alternative therapy or treatment to any medical condition. Please consult with your physician or other healthcare professional.